

Parafriicta[®]

Low Friction Technology™

Low Friction Fabric

Easy-on BOOTEES

(Pair, for single patient use)

To manage the risk of **friction** and **shear-related** pressure ulcers.

To aid in the **prevention** and **management** of heel pressure ulcers.

PFFB-004



Low Friction Fabric

Easy-on BOOTEES (Pair)

- Designed to easily manage the risk of friction and shear-related pressure ulcers
- Designed to aid in the prevention of heel pressure ulcers
- Particularly useful for patients showing early signs of pressure ulcer formation e.g. non-blanching erythema ("red heels")
- For use in a clinical setting
- Are easily removable to inspect vulnerable skin

Latex free

Size	XS	S	M	L	XL
Toe to heel (cm)	20cm to 23cm	23cm to 26cm	26cm to 28cm	28cm to 29cm	29cm to 31cm
Maximum ankle circumference*	22cm	24cm	26cm	28cm	30cm
Product code	PFFB-004XS	PFFB-004S	PFFB-004M	PFFB-004L	PFFB-004XL

*If it is difficult for the user to get a bootee on or off because of swollen ankles, wound dressings etc then consider using a Parafriicta® VELCRO® brand - closure Bootee (PFFB-001)

Instructions for use

- Designed for wear in bed or whilst seated, in combination with pressure relieving equipment.
- The fabric lining the bootee should be in contact with bare skin or wound dressing.
- Select a size according to the table that fits comfortably, based on the measurement from the top of the big toe to the back of the heel.
- Fits either foot. Open the elastic strap before putting on or taking off the bootees. Firmly secure the fastening whilst on the user.
- For single patient use only. Discard bootees sooner if there is heavy soiling, or according to clinical judgement. Dispose of with clinical waste.
- **Important – bootees have a non-slip sole to assist wearers in getting out of bed or chair but should NOT be used as slippers for walking.**

Place barcode label here



Patent nos. GB2403149, EP1641420, CN1812759, JP5117721, AU2004251487 and patents pending

United Kingdom registered design number 6014846

"Parafriicta" is a registered trademark of APA Parafriicta Ltd
VELCRO® is a registered trademark of Velcro BVBA. Used with permission.



This class I medical device is manufactured in the European Union by

Parafriicta®

Low Friction Technology™



APA Parafriicta Ltd

I-Lab Bedford, Stannard Way,
Bedford, MK44 3RZ UK

Tel 0800 0431412 or +44 1234 834525

info@parafriicta.com

www.parafriicta.com

© 2017 APA Parafriicta Ltd