

# Parafricta®

Low Friction Technology™

Low Friction Fabric

## NEONATAL NEST

With Parafricta® fabric side to protect very fragile skin in pre-28 week neonates from damage caused by **friction** and associated **shear**.



PFFNBN-001



## NICU NEONATAL NEST

The Parafri<sup>ct</sup>a<sup>®</sup> NICU Neonatal Nest is designed to protect very fragile skin of pre-28 week neonates against the damage caused by **friction** and associated **shear**.

- Helps to prevent, and aid in the healing of, pressure ulcers and friction lesions.
- Can be washed.

### Instructions for use

- **IMPORTANT—the Neonatal Nest must be washed before first use. SINGLE PATIENT USE ONLY.**
- The neonatal nest is 16.5cm wide x 35.5cm long.
- The nest has Parafri<sup>ct</sup>a<sup>®</sup> fabric on one side that should be in contact with the baby's skin.
- The baby's feet should sit in the pocket at one end and the head rest on the flat end.
- If the nest is not large enough for the baby, consider also using a Parafri<sup>ct</sup>a<sup>®</sup> neonatal sheet on the whole incubator mattress.

### Washing instructions

- The neonatal nest may be washed in a hospital or commercial laundry on a cycle that includes 70°C for up to 10 minutes (sanitising cycle).
- It may also be tumble dried on a low synthetic cycle.
- Dispose of if there is heavy soiling not removed by washing or if there are any signs of damage to the Parafri<sup>ct</sup>a<sup>®</sup> fabric.

Place barcode label here

This class I medical device is manufactured in the European Union by

Parafri<sup>ct</sup>a<sup>®</sup>

Low Friction Technology™

"Little Tew" is a licensed, copyrighted image of Crumble Cottage Ltd © 2015



#### APA Parafri<sup>ct</sup>a Ltd

I-Lab Bedford, Stannard Way,  
Bedford, MK44 3RZ UK

Tel 0800 0431412 or +44 1234 834525

info@parafri<sup>ct</sup>a.com

www.parafri<sup>ct</sup>a.com

© 2016 APA Parafri<sup>ct</sup>a Ltd



Patent nos. GB2403149, EP1641420, CN1812759, JP5117721,  
AU2004251487 and patents pend-ing  
"Parafri<sup>ct</sup>a" is a registered trademark of APA Parafri<sup>ct</sup>a Ltd